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Read these operating instructions carefully, store them for future reference, make them available to other users and follow the instructions.

E

WARNING

- The device is only intended for private use at home, not for commercial use.
- This device may be used by children aged 8 years and older and by persons with reduced physical, sensory or mental capabilities or lack of experience and knowledge if they have been supervised or instructed in the safe use of the device and understand the hazards involved.
- Children must not play with the device.
- Cleaning and user maintenance must not be carried out by children unless they are supervised.
- If the power cord of this device is damaged, it must be disposed of. If it cannot be removed, the device must be disposed of.
- You must never open or repair the device under any circumstances, otherwise proper function is no longer guaranteed. Failure to comply will void the guarantee.

Dear Customer

We are very pleased that you have chosen a product from our range. Please read the operating instructions carefully and keep them in a safe place for future reference.

Kind regards,
Your Waldhausen team

CONTENTS

Delivery contents.....	15
Explanation of symbols.....	15
Get to know the device.....	16
Warning and safety information.....	16
Intended use.....	18
Device description.....	19
Commissioning.....	19
Application.....	19
Super Dandy Go Relax.....	20
Maintenance and proper storage.....	21
What to do if there are problems?.....	21
Technical specifications.....	22
Disposal.....	22

E

DELIVERY CONTENTS

Check the delivery contents for external intactness of the cardboard packaging and completeness of the contents. Before use, make sure that the device and its accessories are not visibly damaged. All packaging material must be removed. If in doubt, do not use it and contact your trusted dealer.

1x massager, 3x interchangeable massage attachments, 1x operating instructions

EXPLANATION OF SYMBOLS

	WARNING Warning notice about risk of injury or dangers to your health.
	CAUTION Safety notice on possible damage to the device/accessories
	INFORMATION Reference to important information.
	The device has double protective insulation and therefore conforms to protection class 2.
	Read the operating instructions.
	For indoor use only.
	Dispose of packaging in an environmentally friendly manner.
	Dispose of the device in accordance with EC Directive WEEE (Waste Electrical and Electronic Equipment).
	Manufacturer
	CE MARKING This product meets the requirements of the applicable European and national guidelines.
	The products verifiably meet the requirements of the technical regulations of the EAEU.

GET TO KNOW THE DEVICE

With this electric massager, you can give your horse a soothing massage effectively and without extra help. Yourself as well, of course. Massages can have a relaxing or stimulating effect and are often used against muscle tension, pain and fatigue. The device offers a powerful, intensive massage for many parts of the horse. The device is not intended for medical or commercial use.

WARNING AND SAFETY INFORMATION

Please read these operating instructions carefully. Failure to observe the following information can cause injury to humans or animals, or damage to property. Store these operating instructions in a safe place and make them available to other users. Hand these operating instructions over when you pass the device on to a new owner.

WARNING

Keep children away from the packaging material. There is a danger of suffocation.

Electric shock

WARNING

As with any electrical device, this massager should be used carefully and thoughtfully to avoid the risk of getting an electric shock.

Therefore, operate the device as follows:

- Only with supply voltage noted on the device
- Never if the device or its accessories are visibly damaged
- Never during a thunderstorm

In the event of defects or malfunctions, switch off the device immediately and disconnect it from the power supply. Do not pull on the supply cable or the device to remove the plug from the socket. Never hold or carry the device by the power cable. See that there is enough distance between the cables and warm surfaces. Do not open the device under any circumstances.

E

If the power cord of the device is damaged, it must be replaced by qualified personnel (not possible for every model).

Make sure that the massager, plug and cord do not come into contact with water or other liquids.

Therefore, use the device as follows:

- Only in dry indoor areas
- Only with dry hands
- Never in the bathtub, in the shower or when the washbasin is full
- Never in swimming pools, whirlpools or in the sauna
- Never outdoors

Never reach for any device that has fallen into water. Disconnect the mains plug immediately.

Repairs **WARNING**

Repairs to electrical devices must only be carried out by specialised personnel. Improper repairs may result in considerable danger for the user. For repairs, please get in touch with customer service or an authorised dealer.

Risk of fire **WARNING**

In the event of improper use or use contrary to these operating instructions, there may be a risk of fire!

Therefore never operate the massager

- longer than 15 minutes (risk of overheating) and let the device cool down for at least 15 minutes before using it again.
- under a cover, such as a blanket, pillow...
- Never leave the device unattended, especially when children are around.

Use

WARNING

The device must be switched off and unplugged after each use and each time before it is cleaned.

E

- Make sure that the moving parts can always move freely.
- Make sure that no hair gets caught in the device.
- Avoid contact with pointed or sharp objects.
- Do not bend the cable.
- Protect the device from heat sources that are too close by, such as ovens, radiators or direct sunlight.

INTENDED USE

This device is designed to massage individual parts of the body. It cannot replace medical treatment. Do not use the massager if one or more of the following warnings apply to your animal. If you are unsure whether the massager is suitable for your horse, ask your veterinarian.

WARNING

- When using the massager on your pet,
- never massage the heart area if it is suffering from heart disease and ask your veterinarian if it can be used for other areas of the body as well,
- never use it on swollen, burnt, inflamed or injured skin and body parts or open wounds,
- never use it during pregnancy,
- do not use it on the head (eyes), on the larynx or other particularly sensitive parts of the body,
- never use it while the animal is sleeping,
- never use it if it is seriously ill or after an operation,
- never use it if it has a fever, cough, runny nose, varicose veins, phlebitis, nerve diseases or acute inflammation,
- do not use it if the animal's medical condition is unclear.

The device is only intended for the purpose described in these operating instructions. The manufacturer cannot be held liable for damage caused by improper or careless use.

DEVICE DESCRIPTION

E

COMMISSIONING

- Remove the packaging.
- Check the device, mains plug and cable for damage.
- Connect the device to the mains supply. The device must be switched off.
- Lay the cable so that nobody can trip over it.

APPLICATION

Make sure the device is off before plugging in the power cord. Switch on the device. Set the desired massage speed with the help of the controller. Change the massage attachments by turning the attachment on the product anticlockwise and replacing it. You can secure the newly affixed massage attachment by turning it clockwise. Guide the device with one or both hands to the body area you wish to massage. You can change the massage intensity by pressing the massage device strongly or less strongly. Do not massage a specific body area for more than 3 minutes; switch to other body areas afterwards.

What can a regular massage do for my horse?

- Relieve tension
- Improve blood circulation → higher muscle performance
- Improve body awareness
- Increase sense of wellbeing
- Strengthen the relationship between horse and human
- Stimulate metabolism
- Activate the self-healing powers
- Able relaxation after exertion

Horses can be massaged at any time

Before work, a massage can help relieve tense muscles. The horse can then use all of its muscles more efficiently during training and, consequentially, build them up faster. The rider can benefit from the horse loosening up faster (massaging does not replace warming up the horse for riding, it only increases muscle performance!) and improved flexibility in all gaits. A massage after work can help relax and reduce muscle soreness.

Duration of a massage: 20 – 90 minutes, depending on the horse's well-being

WARNING

The massage must be perceived as pleasant and relaxing at all times. Stop the massage or change the position of the massager if you feel the massage is painful or uncomfortable for the horse.

SUPER DANDY GO RELAX

Familiarisation phase 1:

- White semi-sphere attachment (10-20 minutes)
- Set the device to the lowest level, increase it whilst massaging and according to how the horse feels, set it back to a low level at the end.
- Carefully massage the upper neck muscles, to the right and left of the spine, shoulder area between shoulder blade and upper arm (triceps), entire croup and hindquarters
- You can basically massage any areas where muscles are clearly visible. Please do not massage the legs, the head, the stomach and be careful of the costal arch area

If the horse expresses significant discomfort in this phase, please consult an osteopath/veterinarian/physiotherapist. Carry out familiarisation phase 1 until the horse shows significant signs of relaxation with this massage.

Familiarisation phase 2:

- Blue needle attachment (10-45 minutes)
- Massage the same regions as in phase 1, but now the deeper muscle layer will be massaged with the needle attachment
- Manage device settings as in phase 1

Familiarisation phase 3:

- White ball attachment, needle attachment, blue trigger point attachment (10-90 minutes)
- Massage the same regions as in phase 1
- Start with the white attachment or needle head; massage the horse in all the areas described
- Pinpoint single points in these areas with your hands
- Massage these points specifically with the trigger point attachment
- End the massage with the white ball head or needle head

In general, the following applies: start with little speed and short times, carefully observing the horse – increase speed and time gradually – always end the massage at a low level.

MAINTENANCE AND PROPER STORAGE

Cleaning

Once the device has been unplugged and has cooled off, wipe it with a slightly damp cloth. You can use a light detergent when doing this.

WARNING

- Unplug the device before cleaning.
- Clean the device only in the manner specified. Under no circumstances should liquids enter the device or the accessories.
- Do not use the device again until it is completely dry.

Storage

Store the massager away from children in a dry, cool place. We recommend storing it in the original packaging.

WHAT TO DO IF THERE ARE PROBLEMS?

Problem	Cause	Troubleshooting
Massage head does not move.	Device not connected to the mains power supply.	Plug in the mains plug and switch on the device.
	Device not switched on.	Switch on the device.
	Overheating protection triggered.	Disconnect the device from the power supply, wait at least 15 minutes, plug it in again and turn it on

E

TECHNICAL SPECIFICATIONS

Supply	200-240 V/50 Hz, consumption: 25 W.
Dimensions	Length 37 cm
Weight	1,000g
Cable length	3m

DISPOSAL

ATTENTION

For environmental reasons, do not dispose of the device in the household waste at the end of its useful life. Dispose of it at a suitable local collection or recycling point. Dispose of the device in accordance with EC Directive – WEEE (Waste Electrical and Electronic Equipment). If you have any questions, please contact the local authorities responsible for waste disposal.

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WALDHAUSEN GmbH & Co. KG